



Test Definition: FANSE

Annatto Seed (Bixa orellana) IgE

Overview

Method Name

Enzyme Immunoassay (EIA)

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Collection Container/Tube:

Preferred: Red top

Acceptable: Serum gel

Submission Container/tube: Plastic vial

Specimen Volume: 0.5 mL Serum

Collection Instructions:

1. Centrifuge and aliquot serum into a plastic vial.
2. Send refrigerated.

Specimen Minimum Volume

See Specimen Required

Reject Due To

| | |
|---------------|--------|
| Gross lipemia | Reject |
|---------------|--------|

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|----------|-------------------|
| Serum | Refrigerated (preferred) | 28 days | |
| | Ambient | 28 days | |
| | Frozen | 365 days | |

Clinical & Interpretive

Clinical Information

Refer to www.eurofins-viracor.com/test-menu/

Reference Values

<0.35 kU/L

Interpretation

| Class | IgE (kU/L) | Comment |
|-------|-------------|----------------------|
| 0 | <0.35 | Below Detection |
| 1 | 0.35-0.69 | Low Positive |
| 2 | 0.70-3.49 | Moderate Positive |
| 3 | 3.50-17.49 | Positive |
| 4 | 17.50-49.99 | Strong Positive |
| 5 | >49.99 | Very Strong Positive |

Performance**PDF Report**

No

Day(s) Performed

Monday through Friday

Report Available

4 to 7 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

86003

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|------------------|--------------------|
| FANSE | Annatto Seed IgE | 50638-6 |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
| Z3074 | Annatto Seed IgE | 50638-6 |
| Z3075 | Class | 103035-2 |