



Test Definition: FGHSU

Gamma-Hydroxybutyric Acid (GHB), Urine

Overview

Reflex Tests

| Test Id | Reporting Name | Available Separately | Always Performed |
|---------|-----------------------------------|----------------------|------------------|
| FGHAC | Gamma-Hydroxybutyric Acid, CF, UR | No | No |

Testing Algorithm

If the GHB Screen, UR is non-negative, then the Gamma-Hydroxybutyric Acid, CF, UR (FGHAC) will be performed at an additional charge.

Method Name

Liquid Chromatography with Tandem Mass Spectrometry (LC/MS/MS)
Gas Chromatography/Mass Spectrometry (GC/MS) (if appropriate)

NY State Available

Yes

Specimen

Specimen Type

Urine

Specimen Required

Collect 10 mL random urine without preservatives. Ship refrigerated in a plastic container.

Specimen Minimum Volume

1.2 mL

Reject Due To

| | |
|-----------|----|
| Hemolysis | NA |
| Lipemia | NA |
| Icterus | NA |
| Other | NA |

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|---------|-------------------|
| Urine | Refrigerated (preferred) | 14 days | |

| | | | |
|--|---------|----------|--|
| | Ambient | 72 hours | |
| | Frozen | 180 days | |

Clinical & Interpretive

Reference Values

Reference Range: Negative

Screening threshold: 5.0 ug/mL

Performance

PDF Report

No

Day(s) Performed

Monday through Sunday

Report Available

5 to 9 days

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

80307

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|-----------------|--------------------|
| FGHSU | GHB Screen, UR | 29868-7 |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
|-----------|------------------|---------------------|

| | | |
|-------|-----|---------|
| Z4226 | GHB | 29868-7 |
|-------|-----|---------|