



# Test Definition: OXYDS

Oxycodone and Metabolites, Serum

## Overview

### Useful For

Monitoring therapeutic efficacy/compliance in serum specimens of patients who are prescribed oxycodone and/or oxymorphone

Detection and quantification of oxycodone, oxymorphone, noroxycodone, and noroxymorphone in serum

### Special Instructions

- [Clinical Toxicology CPT Code Client Guidance](#)

### Method Name

Liquid Chromatography Tandem Mass Spectrometry (LC-MS/MS)

### NY State Available

Yes

## Specimen

### Specimen Type

Serum Red

### Ordering Guidance

This test should only be ordered by healthcare professionals to monitor patients who have been prescribed oxycodone and/or oxymorphone.

### Specimen Required

**Supplies:** Sarstedt Aliquot Tube, 5 mL (T914)

**Collection Container/Tube:** Red top (Serum gel/SST are **not acceptable**)

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 1 mL serum

**Collection Instructions:** Centrifuge and aliquot serum into a plastic vial.

### Specimen Minimum Volume

Serum: 0.5 mL

### Reject Due To

Gross hemolysis	Reject
Gross lipemia	OK

Gross icterus	OK
---------------	----

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	14 days	
	Ambient	24 hours	
	Frozen	28 days	

**Clinical & Interpretive****Clinical Information**

Opiates are the natural or synthetic drugs that have a morphine-like pharmacological action. They are used primarily for relief of pain and often produce physical and psychological dependence. This test is for use by healthcare professionals to determine if patients prescribed oxycodone and/or oxymorphone are receiving the right dose for therapeutic efficacy and/or are compliant.

**Reference Values**

Therapeutic range/Metabolite information (as applicable):

Oxycodone: 10-100 ng/mL

Noroxycodone: Metabolite of oxycodone

Oxymorphone: Metabolite of oxycodone; 1.0-8.0 ng/mL when used as sole therapy

Noroxymorphone: Metabolite of oxymorphone

Cutoff concentrations:

Oxycodone: 1.0 ng/mL

Noroxycodone: 1.0 ng/mL

Oxymorphone: 1.0 ng/mL

Noroxymorphone: 1.0 ng/mL

**Interpretation**

Clinically, the presence of oxycodone, noroxycodone, oxymorphone, and/or noroxymorphone in serum can be used to verify adherence or compliance to prescribed opioids, and discourage drug misuse, abuse, and diversion.

The following adult therapeutic ranges are suggested by literature but must be interpreted alongside other clinical features/test results, symptoms, collection time since last dosage was taken, and signs of abuse/misuse.

Oxycodone: 10-100 ng/mL

Noroxycodone: Not established, it is a metabolite of oxycodone.

Oxymorphone: 1.0-8.0 ng/mL when used as sole therapy, it is also a metabolite of oxycodone.

Noroxymorphone: Not established, it is a metabolite of oxymorphone and downstream metabolite of oxycodone.

**Cautions**

Any aberrant result should be discussed with the patient and could be confirmed by re-testing or using an alternative matrix (eg, urine).

**Clinical Reference**

1. Jannetto PJ, Bratanow NC, Clark WA, et al. Executive summary: American Association of Clinical Chemistry Laboratory Medicine Practice Guideline-using clinical laboratory tests to monitor drug therapy in pain management patients. J Appl Lab Med. 2018;2(4):489-526. doi:10.1373/jalm.2017.023341
2. Langman LJ, Bechtel LK, Holstege CP. Clinical toxicology. In: Rifai N, Chiu RWK, Young I, Burnham C-AD, Wittwer CT, eds. Tietz Textbook of Laboratory Medicine. 7th ed. Elsevier; 2023:chap 43

**Performance****Method Description**

The opiates are extracted from serum using a solvent to precipitate proteins. The supernatant is removed and analyzed by liquid chromatography tandem mass spectrometry.(Unpublished Mayo method)

**PDF Report**

No

**Day(s) Performed**

Tuesday

**Report Available**

2 to 9 days

**Specimen Retention Time**

14 days

**Performing Laboratory Location**

Mayo Clinic Laboratories - Rochester Superior Drive

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test was developed and its performance characteristics determined by Mayo Clinic in a manner consistent with CLIA requirements. It has not been cleared or approved by the US Food and Drug Administration.

**CPT Code Information**

G0480

---

80365 (if appropriate for select payers)

[Clinical Toxicology CPT Code Client Guidance](#)

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
OXYDS	Oxycodone and metabolites, S	In Process

Result ID	Test Result Name	Result LOINC® Value
623456	Oxycodone - Free	12789-4
623457	Noroxycodone - Free	In Process
623458	Oxymorphone - Free	46974-2
623459	Noroxymorphone - Free	In Process