



Test Definition: FALPX

Alprazolam (Xanax)

Overview

Method Name

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

NY State Available

Yes

Specimen

Specimen Type

Varies

Specimen Required

Submit only 1 of the following specimens:

Serum

Draw blood in a plain, red-top tube(s), serum gel tube is not acceptable. Spin down and send 2 mL of serum refrigerated in a plastic vial.

Plasma

Draw blood in a green-top sodium heparin tube(s), plasma gel tube is not acceptable. Spin down and send 2 mL of plasma refrigerated in a plastic vial.

Specimen Minimum Volume

1.0 mL

Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	Serum gel, Plasma gel

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	14 days	
	Ambient	72 hours	
	Frozen	180 days	

Clinical & Interpretive**Reference Values**

5 – 25 ng/mL

Reporting Limit: 2.0 ng/mL

Performance**PDF Report**

No

Day(s) Performed

Monday through Sunday

Report Available

5 to 9 days

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

In-house validated method

CPT Code Information

80346

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FALPX	Alprazolam (Xanax)	59611-4

Result ID	Test Result Name	Result LOINC® Value
FALPX	Alprazolam (Xanax)	59611-4