



Test Definition: FDGTX

Digitoxin, Serum

Overview

Method Name

Immunoassay (IA)

NY State Available

Yes

Specimen

Specimen Type

Serum Red

Specimen Required

*****Must submit one specimen per order. Specimens cannot be shared between multiple orders.*****

Specimen Type: Serum

Collection Container/Tube: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

1. Draw blood in a plain, red-top tube(s). **Serum gel tube is not acceptable.**
2. Centrifuge and send 1 mL of serum refrigerated in a plastic, preservative-free vial.

Specimen Minimum Volume

0.4 mL

Reject Due To

Other	Room Temperature; Serum gel tube (SST), Plasma gel tube (PST)
-------	---------------------------------------------------------------

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	90 days	

Clinical & Interpretive

Reference Values

Reporting limit determined each analysis.

None Detected ng/mL

Therapeutic Range: 10 - 30 ng/mL

Performance

PDF Report

Referral

Day(s) Performed

Monday - Sunday

Report Available

9 to 11 days

Performing Laboratory Location

NMS Labs

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

80299

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FDGTX	Digitoxin	3559-2

Result ID	Test Result Name	Result LOINC® Value
FDGTX	Digitoxin	3559-2