



# Test Definition: FDA1S

Drugs of Abuse (10 panel) and Alcohol Screen,  
Serum

## Overview

## Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FAMCS	Amphetamines Confirmation, S	No	No
FBCS	Benzodiazepines Confirmation, S	No	No
FCANS	Cannabinoids Confirmation, S	No	No
FCMC	Cocaine and Metabolites Confirm, S	No	No
FETCS	Ethanol Confirmation, S	No	No
FMMCS	Methadone and Metabolite Confirm, S	No	No
FOFU	Opiates-Free Unconjugated Conf, S	No	No
FPCCS	Phencyclidine Confirmation, S	No	No
FBCFS	Barbiturates Confirmation, S	No	No

## Testing Algorithm

If appropriate, confirmation testing will be performed at an additional charge.

## Method Name

Enzyme-Linked Immunosorbent Assay (ELISA)

Headspace Gas Chromatography (GC)

## NY State Available

Yes

## Specimen

### Specimen Type

Serum Red

### Specimen Required

**\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\***

**Specimen Type:** Serum

**Collection Container/Tube:** Red top

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 4 mL

**Collection Instructions:**

1. Collect sample using alcohol-free skin preparation.
2. Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable.**
3. Centrifuge and send 4 mL of serum refrigerated in a plastic, preservative-free vial.

### Specimen Minimum Volume

2.00 mL

### Reject Due To

Other	Polymer gel separation tube (SST or PST).
-------	---

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	14 days	

### Clinical & Interpretive

#### Reference Values

Reporting limit determined each analysis.

<u>Test</u>	<u>Result</u>	<u>Units</u>
Ethanol	None Detected	mg/dL
Opiates	None Detected	ng/mL
Cocaine/Metabolites	None Detected	ng/mL
Benzodiazepines	None Detected	ng/mL
Cannabinoids	None Detected	ng/mL
Amphetamines	None Detected	ng/mL
Barbiturates	None Detected	mcg/mL
Methadone/Metabolite	None Detected	ng/mL
Phencyclidine	None Detected	ng/mL
Methamphetamine/MDMA	None Detected	ng/mL
Oxycodone/Oxymorphone	None Detected	ng/mL

**Performance****PDF Report**

No

**Day(s) Performed**

Monday through Sunday

**Report Available**

6 to 20 days

**Performing Laboratory Location**

NMS Labs

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

**CPT Code Information**

80307x2

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
FDA1S	Drug of Abuse 10 and Alcohol Scrn,S	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z5535	Ethanol	20470-1
Z5536	Opiates	8219-8
Z5537	Cocaine/Metabolites	73823-7
Z5538	Benzodiazepines	46976-7
Z5539	Cannabinoids	8172-9
Z5540	Amphetamines	8149-7

## Test Definition: FDA1S

Drugs of Abuse (10 panel) and Alcohol Screen,  
Serum

Z5541	Barbiturates	20421-4
Z5542	Methadone/Metabolite	59705-4
Z5543	Phencyclidine	8236-2
Z5544	Methamphetamine/MDMA	91034-9
Z5545	Oxycodone/Oxymorphone	77732-6