



Test Definition: FMTH

Meconium Methadone Screen with Reflex Confirmation

Overview

Reflex Tests

| Test Id | Reporting Name | Available Separately | Always Performed |
|---------|----------------------------|----------------------|------------------|
| FMTHM | Meconium Methadone Confirm | No | No |

Testing Algorithm

If screen is positive, then the Meconium Methadone Confirmation (FMTHM) will be performed at no additional charge.

Method Name

Immunoassay (IA); Chromatography/with Mass Spectrometry

NY State Available

Yes

Specimen

Specimen Type

Meconium

Specimen Required

Container/Tube: Sterile collection container

Specimen Volume: 3 g

Collection Instructions: Collect specimen into the same sterile collection container until 3 g of meconium have been collected or until the first milk stool appears. When at least 3 g of meconium has been collected, tightly screw on the cap of the collection vial and send specimen ambient.

Note: Specimens from different voids may be pooled if necessary.

Specimen Minimum Volume

1 g

Reject Due To

| | |
|-----------|----|
| Hemolysis | NA |
| Lipemia | NA |
| Icterus | NA |
| Other | NA |

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|---------------------|----------|-------------------|
| Meconium | Ambient (preferred) | 14 days | |
| | Refrigerated | 14 days | |
| | Frozen | 365 days | |

Clinical & Interpretive**Reference Values**

Initial presumptive testing by immunoassay at the following testing threshold:

Methadone: 50 ng/gm

Presumptive positive confirmed by definitive chromatography mass spectrometry (GC/MC or LC/MS-MS)

Performance**PDF Report**

No

Day(s) Performed

Monday through Sunday

Report Available

1 to 6 days

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

80307

Test Definition: FFMTH

Meconium Methadone Screen with Reflex
Confirmation

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|--------------------------------|--------------------|
| FFMTH | Meconium Methadone Scr/Confirm | 43595-8 |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
| Z5603 | Methadone | 43595-8 |