



# Test Definition: PROLX

Fluphenazine (Prolixin), Serum

## Overview

### Method Name

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

### NY State Available

Yes

## Specimen

### Specimen Type

Serum Red

### Specimen Required

**Collection Container/Tube:** Red top

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 3 mL serum

#### Collection Instructions:

1. Centrifuge and aliquot 3 mL of serum into a plastic vial.
2. Send refrigerate.

### Specimen Minimum Volume

0.5 mL

### Reject Due To

Gross Hemolysis	Reject
Gross Lipemia	Reject
Gross Icterus	Reject
Specimens other than serum collected in red top tube	Reject
Plasma or whole blood collected in tubes other than sodium heparin or	Reject

EDTA	
Serum or plasma gel tubes	Reject

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	14 days	
	Ambient	72 hours	
	Frozen	180 days	

**Clinical & Interpretive****Clinical Information**Refer to <https://medtox.labcorp.com/test-menu>**Reference Values**

Reference Range: 1.0-10 ng/mL

**Performance****PDF Report**

No

**Day(s) Performed**

Tuesday through Sunday

**Report Available**

7 to 9 days

**Specimen Retention Time**

2 weeks

**Performing Laboratory Location**

Medtox Laboratories, Inc.

**Fees & Codes****Fees**

- 
- Authorized users can sign in to [Test Prices](#) for detailed fee information.
  - Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
  - Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**CPT Code Information**

80342

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
PROLX	Fluphenazine (Prolixin)	3650-9

Result ID	Test Result Name	Result LOINC® Value
Z1122	Fluphenazine	3650-9