



# Test Definition: FAVI

Hypersensitivity Pneumonitis Avian Panel

## Overview

### Method Name

Immunodiffusion

### NY State Available

No

## Specimen

### Specimen Type

Serum

### Specimen Required

**Preferred:** Red top

**Acceptable:** Serum gel

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 3 mL Serum

#### Collection Instructions:

1. Centrifuge and aliquot 3 mL of serum into a plastic vial.
2. Send refrigerate.

### Specimen Minimum Volume

Serum: 1 mL

### Reject Due To

Gross hemolysis	Reject
Gross lipemia	Reject
Gross icterus	Reject
Empty container	Reject

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	30 days	
	Frozen	365 days	

---

**Clinical & Interpretive****Clinical Information**

Refer to

[www.mcw.edu/departments/pediatrics/divisions/asthma-allergy-and-clinical-immunology/patient-care/allergy-immunology-diagnostic-lab-center](http://www.mcw.edu/departments/pediatrics/divisions/asthma-allergy-and-clinical-immunology/patient-care/allergy-immunology-diagnostic-lab-center)

**Reference Values**

The following antigens will have results reported:

Pigeon Sera

Pigeon DE

Cockatiel

Parakeet

Parrot

Results are reported as:

Negative

Weakly Positive

Positive

Strong Positive

This result must be correlated with patient's clinical response and should not solely be considered in the diagnosis.

**Performance****PDF Report**

No

**Day(s) Performed**

Thursday and Friday

**Report Available**

18 to 20 days

**Performing Laboratory Location**

Medical College of WI Allergy-Immunology MACC Fund Research Ctr.

**Fees & Codes****Fees**

- 
- Authorized users can sign in to [Test Prices](#) for detailed fee information.
  - Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
  - Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**CPT Code Information**

86331x5

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
FAVI	Avian Panel	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z0289	Pigeon DE	Not Provided
Z0285	Parakeet	Not Provided
Z0286	Cockatiel	Not Provided
Z0287	Parrot	Not Provided
Z0288	Pigeon Sera	Not Provided