



# Test Definition: FAEAB

Anti-Enterocyte Antibodies

## Overview

### Special Instructions

- [Anti-Enterocyte Antibody \(AEA\) Clinical Form](#)

### Method Name

Indirect Immunofluorescence

### NY State Available

No

## Specimen

### Specimen Type

Serum Red

### Necessary Information

[Anti-Enterocyte Antibody \(AEA\) Clinical Form](#) is required. Complete the form and submit with the specimen. **Testing will not proceed without this required form.**

### Specimen Required

**Collection Container/Tube:** Red top

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 2 mL Serum

#### Collection Instructions:

1. Centrifuge and aliquot 2 mL of serum into a plastic vial.
2. Send frozen.

### Forms

[Anti-Enterocyte Antibody \(AEA\) Clinical Form](#) is required.

### Specimen Minimum Volume

Serum: 1 mL

### Reject Due To

All specimens will be evaluated by the processing and performing laboratories for test suitability

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Frozen		

## Clinical & Interpretive

### Clinical Information

Refer to [www.testmenu.com/chop](http://www.testmenu.com/chop).

### Reference Values

IgG: Negative

IgA: Negative

IgM: Negative

## Performance

### PDF Report

No

### Day(s) Performed

Batched weekly

### Report Available

9 to 11 days

### Performing Laboratory Location

Children's Hospital of Philadelphia

## Fees & Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

### Test Classification

These tests were developed and their performance characteristics determined by the Pathology Department at The Children's Hospital of Philadelphia. They have not been cleared or approved by the U.S. Food And Drug Administration. The FDA has determined that such clearance or approval is not necessary. These tests are used for clinical purposes. It should not be regarded as investigational or for research. This Laboratory is certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA) as qualified to perform high complexity clinical laboratory testing.

### CPT Code Information

88346

88350 x 2

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
FAEAB	Anti-Enterocyte Antibodies	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z1700	Anti-Enterocyte Antibodies	Not Provided
Z1687	Dilution of Serum	Not Provided
Z1688	IgG	Not Provided
Z1689	IgA	Not Provided
Z1690	IgM	Not Provided
Z1691	Signed	Not Provided